
# Poise™ • Ejector Lancing Device with Armed Indicator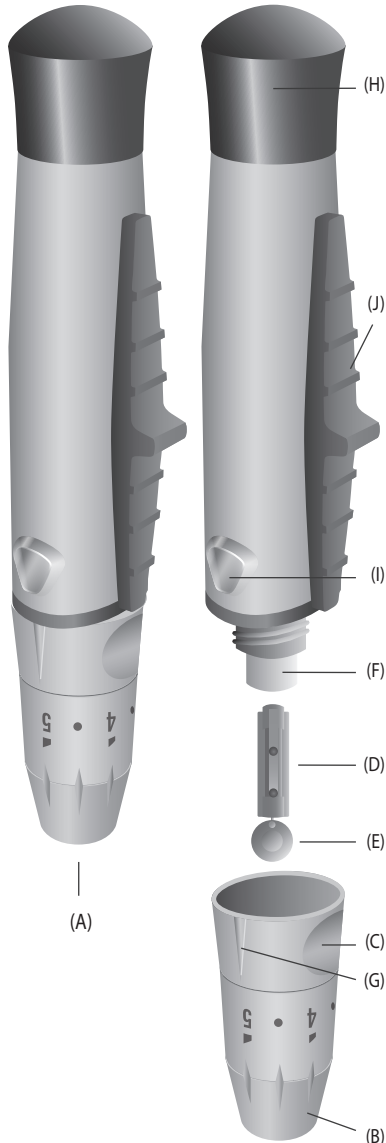

## English: [www.Intrinsyk.com/Poise-LA-IFU-EN.pdf](http://www.Intrinsyk.com/Poise-LA-IFU-EN.pdf)

### LANCING DEVICE INSTRUCTIONS: (numbers below match up with numbered graphics)

1. Hold the Poise device (A) firmly with one hand and remove the end cap (B) by holding the cap base (C) and turning it counter-clockwise.
2. Hold lancet (D) firmly in one hand, twist and pull off protective lancet cover (E) with the other hand.
3. Carefully insert lancet (D) into the lancet holder (F) until it comes to a stop. **NOTE: Do not touch the lancet tip.**
4. Replace the end cap (B) by holding the cap base (C) and turning it clockwise onto the Poise device (A).
5. Select the skin penetration depth by turning the end cap (B) until the desired number lines up with the arrow (G) on the cap base (C). The Poise device offers 5 levels of skin penetration in half level increments: up to 1.5 for soft skin; 2-3.5 for average skin; and 4-5 for thick or calloused skin.
6. Pull back on the Poise cocking mechanism (H) located at the back of the device until it comes to a stop (you should hear a click when device is cocked). The cocking mechanism will return to its original position upon release. When the device is cocked, the armed indicator in the window of the release button (I) will turn RED.
7. Place Poise firmly against the finger and press the release button (I) to lance the finger and obtain a blood sample. The lancet will return to its original position after use. After use, the armed indicator window will return to clear.
8. After finishing your blood test, remove the Poise end cap (B) and position lancet directly over appropriate container. Firmly holding the device (A) with one hand, push the Ejection slide (J) in the direction of the lancet (D). The used lancet will safely drop into approved container.

Additional information about your Poise Lancing Device on reverse side. >

## Español: [www.Intrinsyk.com/Poise-LA-IFU-ES.pdf](http://www.Intrinsyk.com/Poise-LA-IFU-ES.pdf)

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